Fixierband

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EN Directions for use for Attachment band

Please read these operating instructions in full before applying the attachment band for the first time. It contains important information. Please store the instructions for use in a safe place. They are part of the product.

Intended use

The attachment band helps support the fixing of glucose sensors to the upper arm for everyday use as well as for physical activities, in both air and water (even in sea water). This attachment band can be reused as many times as required, provided it is not damaged. It is designed for use on healthy, undamaged skin. It can be used all day, just not when sleeping.

Indications

For when a support is required for affixing a glucose sensor, such as for physical activities, including those in water (fresh water and salt water).

User / patient groups

The attachment band can be used by diabetics who must use a CGM sensor. They include children with an arm circumference greater than 15 cm and those requiring care where a guardian can apply and remove the attachment band.

Contraindications

The attachment band must not be used in the following cases:

- Skin and tissue injuries on and around the site it is applied that do not allow coverage under pressure for longer periods of time, such as damaged skin, pressure-sensitive tissue, rashes, lymphoedema, haematomas.
- Situations that may result in the attachment band being moved due to involuntary movements or external influences, such as when sleeping in bed or when playing contact sports.
- A pronounced conical shape of the upper arm from top to bottom, as the attachment band will slip down.
- Allergic reactions to the materials used.

Warnings

- Use of the fixing is at your own risk, no liability can be accepted for sensor damage/loss due to improper use.

- The attachment band must only be used on healthy, intact skin otherwise infections may occur. Do not wear the attachment band if it comes into contact with scratches, cuts, burns, sunburn or any form of skin rash or irritation.

- The attachment band must only be worn by people with no sensory restrictions on the upper arm, where the sensor is positioned. Otherwise there may be too much contact pressure or skin damage arising from improper use which may go unnoticed and it could lead to issues such as inflammation.

- The attachment band may lead to incorrect measurements on the glucose sensor if used incorrectly. For this reason, follow the instructions closely when applying the attachment band. Monitor blood sugar levels in the blood if values are high or low.

- Use on heavily moisturized or anointed skin may result in the attachment band slipping, which means the sensor may become loose or incorrect measurements may occur.

- Only use and store the attachment band within the minimum and maximum temperature range stated in the technical data, otherwise the band, adjuster or sensor holder may be damaged, which leads to a loss of function.

- Do not attach any other objects with this attachment band. This may lead to the band slipping, coming off, or incorrect measurements.

- The attachment band must neither be too tight, nor too loose. If it is too loose then there is a risk that the sensor slips. If it is too tight then crushing or skin redness may occur. If any form of irritation occurs, remove the attachment band immediately.

- If wearing the attachment band triggers allergic reactions, then take off the attachment band immediately and do not wear it again. The band may have been cleaned with an agent that triggers the allergic reaction. In this case, clean the band with an agent that does not trigger allergic reactions.

- Using this band over the maximum temperature of 50° C may lead to skin burns and a loss of function on the attachment band.

Safety precautions

- The attachment band must only be used if a glucose sensor can be safely applied. There is otherwise the risk that the attachment band is not used correctly due to a lack of knowledge and skills, which may lead to sensor damage or the sensor coming loose.

- Take care when applying the attachment band so that it is correctly in place and that the sensor is not prematurely moved, either due to hasty application or incorrect positioning.

- Only use stated cleaning agents and processes to avoid skin irritation or allergic reactions to the cleaning agents.

- Do not apply external forces to the attachment band so that it does not slip, and therefore risk that either the sensor becomes loose or provides incorrect measurements.

- If the band loses elasticity, there is damage to the sensor holder or to the adjuster then the attachment band must no longer be worn as there is a risk that the attachment function does not work properly, leading to the sensor slipping, generating false measurements, or coming off completely.

Dealing with cases of skin irritation

The attachment band consists of a strap that is 83% cotton and 17% elastane. It contains no latex, meaning there is less risk of skin irritation. Persons with extremely sensitive skin may suffer from skin rash, itchiness or irritation. In this case, remove the attachment band and do not wear it again.

Maintenance

The attachment band does not require any special maintenance. Use a mild detergent and wash the entire band by hand to clean off any dirt. It can also be washed in the washing machine at 30°C. Use a laundry bag so that the laundry and the attachment band are not damaged.

Storage

The attachment band should be stored in a cool, dry place, away from direct sunlight. Direct sunlight and direct contact with water must be avoided during storage.

Contacting the manufacturer or the relevant authorities

If something serious has happened that is linked to the product, then please contact us at Diasticker and/or NHS National Health Service. A serious event is any event that directly or indirectly leads, could have led, or could lead to one of the following incidents:

- death of a patient, user, or another person;
- serious temporary or permanent effects on the health of a patient, a user, or another person;
- a serious risk for public health.

Disposal

At the end of its operating life, dispose of the attachment band either as domestic waste or in the recycling bin.

Customer Services

Please contact the retailer or us, **email:** info@diasticker.de, **phone:** +49 951/70045771, **homepage:** <https://www.diasticker.de/kontakt/> **Website for downloading these operating instructions:**

www.diasticker.de/IFU_Fixierband

Website for downloading the declaration of conformity:

www.diasticker.de/declaration_of_conformity

Technical details

Arm circumference sizes:

- Size S: 15 – 25 cm
- Size M: 25 – 35 cm
- Size L: 35 – 45 cm

Band width: 25 mm

Weight: 11g (size S) - 12g (size M) - 13g (size L)

Temperature range: -10°C to +50°C

Strap material: Polyester, elastodiène, Elastane

Sensor holder material: plastic suitable for medical purposes

Shelf life: The shelf life of the attachment band depends on the type and frequency of use.

Risk class: I, Rule: I, from Appendix VII, EU MDR

Wearing the attachment band

The attachment band is suitable for use in sporting activities. However, it should not be

worn for contact sports such as wrestling. It is ideal for going swimming, jogging in the woods, or going to the gym. Afternoons in the sauna or exuberant playing around are also fine. You can even wear the band in the shower without issue. The band dries very quickly.

Removing the attachment band

To remove the attachment band from the sensor once again, simply lift the plastic holder up from off the sensor, and pull the attachment band downwards towards the hand and below the sensor.

Application of the attachment band

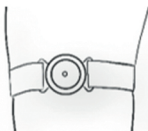
1.

2.

3.

Application of the attachment band

Check the attachment band for signs of damage or ageing before application. The sensor holder and the adjuster must not have any cracks. The strap must be stretchy and not be worn out.

Step 1

Apply the attachment band below the sensor. CAUTION: Never adjust the size directly on the sensor as this may move out of position as a result. The attachment band must not be too loose on the arm. However, it also must not be so tight that it leaves marks on the skin.

Step 2

Depending on the fit, take off the attachment band once more and adjust the size accordingly.

Step 3

When the correct size is set, apply the attachment band below the sensor once again. The sensor holder is then lifted over the sensor. Ensure that the strap is in position and does not pull the sensor downwards.

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Diasticker GmbH & Co. KG