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**en** Supplement for Instructions for use

Alternative option to refill the mylife YpsoPump Reservoir with the transfer adapter:

→ Start the process by initially filling a new reservoir, according to the **handling steps 1–9 from the instructions for use**, packed in this product. Then follow the instructions below:

- Remove the reservoir from the transfer adapter and **keep the adapter on the insulin vial until this filling process is completed**. Only use the transfer adapter for one filling/refilling process.
- Twist off the plunger rod from the newly filled reservoir (**do not dispose of the green plunger rod**).
- Mark the new filled reservoir with a permanent marker (date of filling) and put it in the fridge.
- Remove the empty reservoir from your pump.
- Screw in the green plunger rod into the plunger of the empty reservoir.
- Clean the septum of the empty reservoir with an alcohol swab.
- Refill the reservoir according to the handling steps 4, 7, 8 and 9 from the instructions for use, packed in the product.
- Remove the refilled reservoir from the transfer adapter.
- Twist off the plunger rod, the transfer adapter and dispose them in accordance with local laws.
- Put the refilled reservoir back in the pump.
- When the refilled reservoir is empty, take the new filled reservoir from the fridge.

For further contact details, please visit www.mylife-diabetescare.com

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